## MultiStatic

Correction Joint

Please read these instructions carefully before using the product. Pay particular attention that the safety instructions are followed.

**Usage guidelines**

- The product may only be fitted by or under the supervision of a competent professional
- Special precautions must always be taken for patients with a visual impairment, cognitive limitations and/or reduced sensibility in the upper and/or lower extremity.
- Changes, modifications and adjustments to the product that are not described in these instructions, are done at the sole responsibility of the person carrying them out.

**Intended use of the product**

The MultiStatic corrective Joint is used for treatment of correctable contractures, both for children as well as adults. The Joint enables fast, efficient and adequate static treatment of contractures according on the High Load Brief Stretch (HLBS) principles. The angle of the joint is set by an eccentric axis.

The corrective joint is to be used exclusively in orthotics for the upper or lower extremities and should not be weight bearing.

We strongly advise to use the FreeMotion companion joint next to the MultiStatic corrective joint at the contralateral side of the affected joint to ensure sufficient torsional stiffness of the orthosis.

**Note:**

- No weight bearing
- Fall injury, not suitable for walking

MultiStatic is suitable for static contracture treatment with a neurological or orthopedic origin.

**Neurological indications:**

- Apoplexy (CVA)
- Cerebral palsy
- Cranial Brain Trauma
- Dystrophies
- Multiple sclerosis
- Status after spinal cord injury
- Spina Bifida

**Orthopedic indications:**

- Amputation
- Fractures
- Status after ligament rupture
- Status after joint surgery

**Contraindications:**

- Structural Contractures as:
  - Ankylosis
  - Fibrosis
  - Ossification
- Muscle and bones deformation
- Poor blood circulation

**General safety instructions**

Risk of injury as a result of incorrect use; the professional should verify correct functioning of the joints prior to delivery of the orthosis to the user. The patient and his/her care giver(s) should also be instructed on donning and doffing of the orthosis and have the joint functionality explained to them. It should also be made clear to them that the spring tension setting should never be changed by anybody else than the professional or therapist, unless the treating physician decides otherwise. The joints should not come into contact with water.

**Comment**

Specialists opting for and/or fitting and customize this product, should use their professional judgment in choosing the product, fitting it and adequately educating the patient or care giver so as to minimize potential risks for each individual patient. These risks may include the mentioned contraindications, but may also be associated with the unique patient characteristics or circumstances of the patient’s care giver.

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