

World Leader in Carbon Composite AFO Technology



The quantum leap in AFO technology

Introduced in 1997, ToeOFF®, the first Allard AFO, is now recognized as a quantum leap in AFO technology. Today hundreds of thousands of individuals with gait impairment now enjoy a higher quality of life, thanks to the stability and dynamic response provided when wearing one of our many carbon composite orthoses. More experience than any company in the industry and our commitment to providing only the highest quality product and customer service continue to position us as the world leader in carbon composite AFO technology. In pursuit of our drive to expand our “Support for Better Life”, our product development team aggressively researches new materials, technology, and product designs that will extend the opportunity for improved gait and walking capacity to more and more individuals with lower extremity deficiencies.



Anterior Design

Allard AFOs extend up from the footplate onto the anterior surface of the leg to avoid pressure on the calf muscles and Achilles tendon. This allows for a dynamic floor reaction response to assist with propulsion and knee extension

Lateral Strut

The strut is placed on the lateral side of the footplate to create stability and function. It also allows for more functional use by bilateral users.

Light Weight

The composition of Carbon fiber, Fiberglass and Kevlar produces an extremely lightweight yet structurally superior design, which increases user compliance.

Open Heel

The open heel design allows the calcaneus to invert and evert to activate the natural biomechanical chain reaction to occur to achieve a closer to normal gait and eliminates uncomfortable pressure on the back of the leg or heel.

Comfortable

As the shin plate comes on the anterior surface of the leg, it provides a more comfortable design for the user.

Thin

The extreme thinness of the product makes it light and almost invisible under trousers.

Dynamic Footplate

The unique layup and the shape of the footplate contributes to a more dynamic, functional, and fluid gait pattern.


Fits in Shoe

The thinness of the footplate often avoids the need to increase the shoe size.

Graded Stability

To be able to accommodate different users' needs, the sizes and products are graded in both stability and dynamic response.

Product Selection Guide

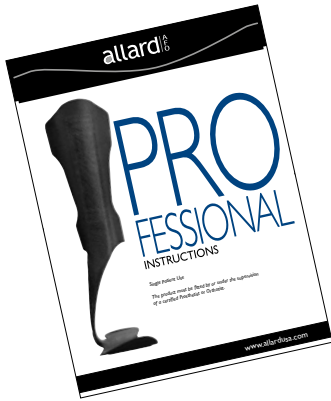


Further information about intended use, indications and contraindications, and suggested modifications to assure optimum alignment and patient comfort can be found in the Professional Instructions packed together with each Allard AFO or can be downloaded from allardusa.com. You can also view our 8-step videos on our website.

The scales on this page and chart on following page are provided as general guidelines only. Each individual, based on their unique deficits, gait patterns, proprioceptive response, and lifestyle needs, will respond differently to any orthotic device. This will influence the function and compliance results.

Allard AFO Product Rigidity Scale

Each Allard AFO offers different rigidity to provide the amount of support needed without over-bracing. The chart below shows the variance in the amount of support each AFO offers. For example, Ypsilon® Flow½ would be considered for patients requiring minimum support while BlueROCKER® would be considered for those requiring the most support.

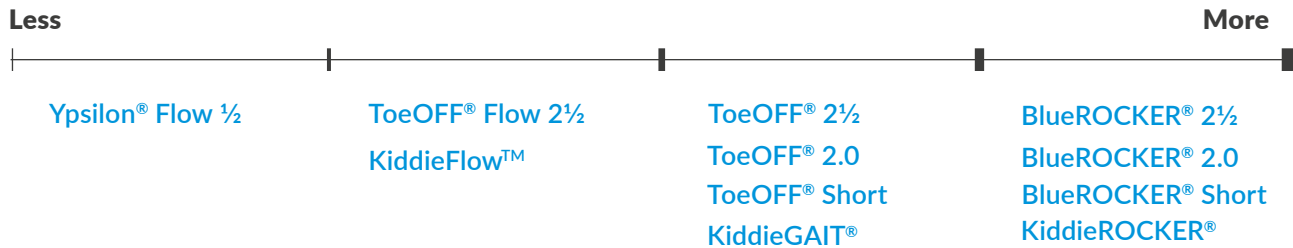

Allard AFO function scale

As an example, Ypsilon® FLOW½ allows for the most ROM while BlueROCKER® offers the least. On the other hand, if you look at stability, BlueROCKER® offers the most whereas Ypsilon® FLOW½ will provide the least stability.


	Ypsilon® Flow ½	ToeOFF® Flow 2½ KiddieFlow™	ToeOFF® 2½ ToeOFF® 2.0 ToeOFF® Short KiddieGAIT®	BlueROCKER® 2½ BlueROCKER® 2.0 BlueROCKER® Short KiddieROCKER®
Allows Range Of Motion	●●●●○	●●●○	●●○○○	●○○○○
M-L Stability	●●○○○	●●●○	●●●●○	●●●●●
A-P Stability	●●○○○	●●●○	●●●●○	●●●●●
Dorsiflexion Assist	●●○○○	●●●○	●●●●○	●●●●●
Spasticity Control	●●○○○	●●●○	●●●●○	●●●●●
Proximal Control	●●○○○	●●●○	●●●●○	●●●●●

Allard Muscle Weakness Guide

These charts are based on R&D studies, practitioner, and patient feedback. When patient's anatomical or function needs fall out of the parameters of the Product Rigidity or Function Scale, Allard also offers to fabricate Custom Carbon Composite AFOs. Ask Customer Service for ordering information.

Muscle function None or limited function ● Moderate function ● Full function ●

LIMITED Proximal weakness

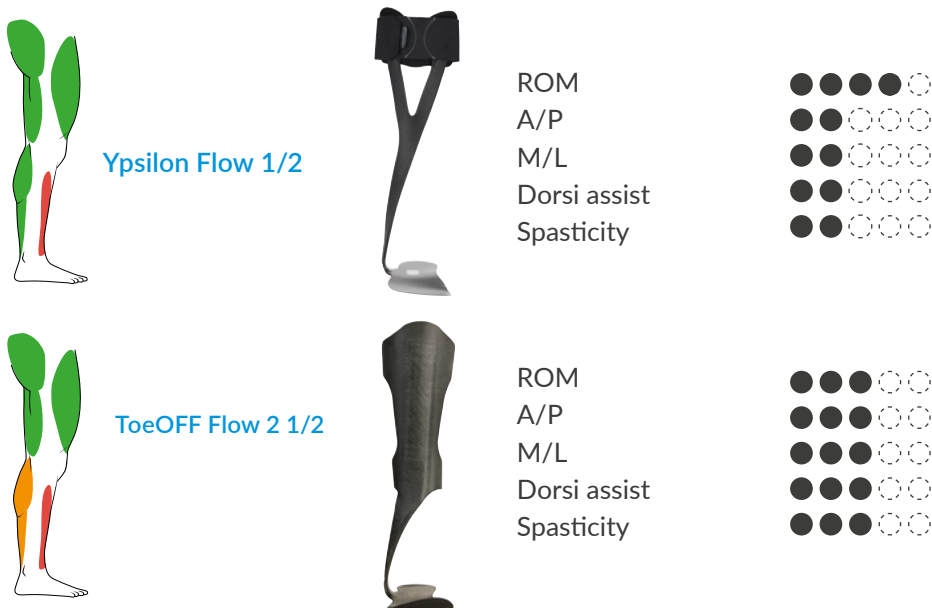

MORE Proximal weakness



The “Flow” technology

We applied Allard’s 20+ years of AFO carbon composite experience and advances in technology to create a new proprietary formula to meet the functional and comfort needs of even more individuals.

The “Flow” benefits

- Smoother transition (flow) throughout the gait cycle
- Increased ROM in sagittal plane
- Accommodates lower shoe heel heights
- Accommodates gentler contours of shoe insoles
- More clearance for forefoot in the shoe toe box
- Easier to customize to meet alignment and biomechanical needs



ToeOFF® Flow 2½

Recommended for mild to moderate ankle instability and mild to moderate proximal deficits.

- Provides added rigidity to stabilize the unstable ankle and provide added control of proximal deficits.
- FLOW offers greatest ROM in sagittal plane and smoother transition throughout the gait cycle.
- To better accommodate today's shoe fashions this orthosis has 7mm heel height.
- A more organic shape with shorter wings.

ToeOFF® Flow 2½ is suggested for patients with:

- Mild proximal deficits
- Mild spasticity
- Limb proprioception deficit
- Functions well for higher activity levels



Functional Guidelines

Allows Range of Motion



M-L Stability



A-P Stability



Dorsiflexion Assist



Spasticity Control



Proximal Control



Scan or click for sizes
and measurements!



Ypsilon® Flow ½

Ypsilon® Flow ½ was designed for the stable ankle to provide dynamic toe-off assistance while allowing natural ankle movement. ComfortPAD™ included.

- Allows more medial, lateral and rotational ankle movement, which will provide an opportunity for muscles, tendons and ligaments to function and strengthen
- FLOW offers greatest ROM in sagittal plane and smoother transition throughout the gait cycle.
- To better accommodate today's shoe fashions this orthosis has 7mm heel height.

Ypsilon® Flow ½ is suggested for patients with:

- Weak dorsiflexors
- Peroneal nerve injury
- No/mild spasticity
- Sensory Nerve Injury
- Functions well for higher activity levels



Functional Guidelines

Allows Range of Motion



M-L Stability



A-P Stability



Dorsiflexion Assist



Spasticity Control



Proximal Control



Scan or click for sizes
and measurements!



ToeOFF® 2.0, ToeOFF® 2½, ToeOFF® Short

Designed for mild to moderate ankle instability and mild to moderate proximal deficits.

- Provides added rigidity to stabilize the unstable ankle and provide added control of proximal deficits.
- Compared to FLOW these have increased stability in the sagittal plane.

Suggested for patients with:

- Mild to moderate proximal deficits
- Mild to moderate spasticity
- Limb proprioception deficit

Functional Guidelines

Allows Range of Motion



M-L Stability



A-P Stability



Dorsiflexion Assist



Spasticity Control



Proximal Control



Scan or click for sizes and measurements!

ToeOFF® 2.0



- A more organic shape with shorter wings
- 12-15mm heel height
- Full toe lift
- Starter SoftKIT included & all MikroFIX Pre-Applied

ToeOFF® 2 ½



- A more organic shape with shorter wings
- 7mm heel height
- Less toe lift
- Starter SoftKIT included & all MikroFIX Pre-Applied

ToeOFF® Short



- Full wings
- 7mm heel height
- Shorter height
- Wider at metatarsal heads
- Flatter footplate contour

BlueROCKER® 2.0, BlueROCKER® 2½, BlueROCKER® Short

Designed for moderate to severe ankle instability and mild to severe proximal deficits.

- The extra stability will improve both balance and posture and give the wearer greater security, especially individuals with weak quadriceps and calf muscles.
- Provides maximum rigidity to stabilize the unstable ankle and provide added control of proximal deficits.

Suggested for patients with:

- Moderate to severe proximal deficits
- Moderate to severe spasticity
- Partial foot amputation
- Weakness or impairment in multiple leg muscle groups
- The need to wear bilateral AFOs
- Reduced knee and hip control

Functional Guidelines

Allows Range of Motion



M-L Stability



A-P Stability



Dorsiflexion Assist



Spasticity Control



Proximal Control



Scan or click for sizes and measurements!

BlueROCKER® 2.0



- A more organic shape with shorter wings
- 12-16mm heel height
- Full toe lift
- Starter SoftKIT included & all MikroFIX Pre-Applied

BlueROCKER® 2 ½



- A more organic shape with shorter wings
- 7mm heel height
- Less toe lift
- Starter SoftKIT included & all MikroFIX Pre-Applied

BlueROCKER® Short



- Full wings
- 7mm heel height
- Shorter height
- Wider at metatarsal heads
- Flatter footplate contour

KiddieFLOW™, KiddieGAIT®, KiddieROCKER®

KiddieFLOW™, KiddieGAIT® and KiddieROCKER® offer a carbon composite framework for orthotists to build an orthosis that will provide needed stability and dynamic response to encourage functional heel-to-toe gait.

- Anterior support provides opportunity to enhance proprioceptive response and adds stability to help manage knee extension/flexion moment.
- Soft and dynamic heel rocker which creates softer loading to reduce knee flexion moment.
- Lightweight functional support allows more fluid and efficient less tiring gait.

Suggested for patients with:

- Footdrop
- Gait deviations secondary to proprioceptive deficit
- Unstable or low-tone gait
- Toe-Walking with no midfoot collapse
- Low-tone crouch gait (except KiddieFLOW™)
- Partial foot amputation (KiddieROCKER® preferred)

KiddieFLOW™, KiddieGAIT® and KiddieROCKER® should always be combined with an additional orthotic, designed to control the position of the foot.



KiddieGAIT® with SMO



KiddieGAIT® with UCB



KiddieGAIT® with Surestep™



Scan or click for sizes
and measurements !



KiddieFLOW™



Offers generous ROM in sagittal plane and smoother transition throughout the gait cycle.

- A more organic shape with shorter wings
- 5-7 mm heel height
- Full toe lift

Functional Guidelines

Allows Range of Motion



M-L Stability



A-P Stability



Dorsiflexion Assist



Spasticity Control



Proximal Control



KiddieGAIT®



Compared to KiddieFlow KiddieGAIT has increased stability in sagittal plane.

- A more organic shape with shorter wings
- 4-7 mm heel height
- Less toe lift

Functional Guidelines

Allows Range of Motion



M-L Stability



A-P Stability



Dorsiflexion Assist



Spasticity Control



Proximal Control



KiddieROCKER®



Offers more rigid orthotic control, primarily for bilateral foot drop patients and those with more involved pathologies.

- Full wings
- 5-7 mm heel height
- Shorter height
- Wider at metatarsal heads
- Flatter footplate contour

Functional Guidelines

Allows Range of Motion



M-L Stability



A-P Stability



Dorsiflexion Assist



Spasticity Control



Proximal Control



Interface Options



SoftKIT™

SoftKIT™ is our standard 6 mm soft foam interface system which includes 2 vertical tibia pads. May be applied to all Allard AFOs except Ypsilon Flow ½. All 2.0 and 2 1/2 Allard AFOs have a Starter SoftKIT included.



ComfortKIT™

ComfortKIT™ is our premium 9 mm memory foam interface. It is designed for the patient that requires added cushioning and a more intimate fit due to the shape of the tibia, skin condition, diabetes and/or activity level. Available for most Allard AFO's.



CoverKIT™

CoverKIT™ is another interface innovation for our 2.0 & 2½ models. The interface has a thin textile front to form a sleeve and an inside soft interface that protects the skin and at the same time keeps the brace in position.



SoftSHELL™

SoftSHELL combines the soft interface system of SoftKIT with a fun-print textile cover, made just for kids!

Fits KiddieFLOW™, KiddieGAIT®, & KiddieROCKER®.



Scan or click for Accessories & Replacement Parts.



Accessories



Foot Orthotic Shell

These prefabricated foot orthotics help to better position and align the pronated foot, while providing a comfortable and shock absorbing interface between the footplate and foot. Allard FOS are fabricated with EFM (Extra Firm Material) and Microcell Puff Interface.



For Pes Valgus Control

T-Strap

T-Strap pulls the the talocrural and subtalar joints laterally to position them back underneath the tibia.

Quick and easy to apply: Distal end attaches to bottom of footplate, proximal "T" wraps around ankle and attaches to the Allard AFO's lateral strut.



Wedge Kit

Kit that includes lateral wedge and heel wedges. Posting wedges are used for correcting or adjusting medial & lateral slopes. Pressure Sensitive Adhesive (PSA) Backed EVA Foam. All wedges are Shore A60 Durometer.



Scan or click for Accessories & Replacement Parts.

Patient satisfaction

Allard offers a unique Patient Satisfaction Guarantee and a generous warranty policy. Below is summary only, please ask Customer Service for additional details.

Note: Due to the uniqueness of custom products, the Patient Satisfaction Guarantee does not apply to Allard Custom AFOs.

- If the wearer, therapist, physician, or orthotist is not satisfied with the performance of the product initially fitted, Allard USA will, at the provider's discretion, provide a FREE alternative product for the wearer to try.
- ALL Patient Satisfaction return requests must be received by Allard within 30 days of initial fitting.

Allard AFO Warranty

Allard return policy for Allard AFO products assumes that fitting has been done correctly, adjustments have been carried out according to Professional Instructions supplied with each product and Patient Instructions have been reviewed with patient and provided to him or her or caregiver to take home. Allard grants warranty for defects in material and workmanship. Warranty registration must be submitted within 10 days of initial fitting to validate warranty.

Allard Adult AFOs EXCEPT Custom

- Allard will provide a replacement Allard AFO of equal value to the provider.
- Choice of specific product and product size is at the discretion of the provider.
- Replacement is limited to two orthoses for the same wearer that are returned within one year from the date of the initial fitting of the Allard AFO.

Allard Adult Custom AFOs

- Allard will provide a replacement Allard Custom AFO of equal value to the provider, provided the replacement does not require change in position of the strut or change in heel height (unless the change is required due to a fabrication error).
- There will be an additional charge if the replacement requires a change in 7 properties specified in the original order, for example changes in A-P and/or M-L stiffness.
- Replacement is limited to one orthosis for the same wearer that is returned within one year from date of initial fitting.

Allard Pediatric AFOs (KiddieFLOW™, KiddieGAIT® & KiddieROCKER®)

- Allard will provide a replacement Allard AFO of equal value to the provider.
- Choice of specific product and product size is at the discretion of the provider.
- Replacement is limited to one orthosis for the same wearer that is returned within six months from the date of the initial fitting of the Allard AFO.

Dralla

Giving individuals with physical challenges an unforgettable day!

Introducing Dralla Foundation, a non-profit organization that provides grants to other non-profits that align with their mission. The mission of Dralla Foundation is to make unforgettable days possible for adults and children with physical challenges by partnering with non-profit organizations to fund their most meaningful and memorable programs.

Visit www.dralla.org to learn more about the organization, to submit a grant request, and to see pictures, testimonials, and videos of some of the unforgettable days made possible. Allard proudly donates a portion of every AFO sold to fund Dralla's partnership events.



Photo courtesy of Adaptive Sports Program New Mexico



Photo courtesy of Adaptive Sports Foundation



Photo courtesy of Daring Adventures

Learn more about how an Allard AFO can become a life changer!

MEET THE TEAM



get back **UP** today

Allard USA proudly advocates for those living with mobility issues by supporting the "Get Back UP Today" grassroots movement, an international community of people with foot drop and similar conditions.

Led by volunteer Co-Captains, getbackUPtoday is proof that mobility challenges do not have to alter one's quality of life. For your patients who may be struggling with "getting back up" due to their physical challenge, we hope you will encourage them to visit www.getbackUPtoday.com to read inspiring stories, find links to other resources and to join a world-wide team of diverse individuals living their best lives.



Assessment tools

Allard also offers AFOs that are designed to be used as initial assessment tools to evaluate functional improvement, patient acceptance and as a gauge to decide which Allard AFO will work best for the patient as well as what modifications will be necessary to optimize the patient's gait pattern. Those are labeled "Not for resale".



KiddieFLOW™
Youth Sizes



KiddieGAIT®
Baby Sizes



KiddieGAIT®
Youth Sizes



KiddieROCKER®
Youth Sizes



Ypsilon® FLOW½



ToeOFF® FLOW 2½
ToeOFF® 2½
ToeOFF® 2.0



ToeOFF® SHORT



BlueROCKER® SHORT



BlueROCKER® 2½
BlueROCKER® 2.0

Offered individually or in convenient "6-PACKS" (sizes S, M and L – left and right).



Scan or click for sizes
and measurements!

Support for Better Life

Everyone should be able to live their life to the fullest, regardless of their mobility challenges. With innovative solutions developed in close collaboration with healthcare professionals and patients, we strive to provide Support for Better Life.

allard

Allard USA Inc.

300 Forge Way, Suite 3, Rockaway NJ 07866-2056, USA | 888-678-6548 | info@allardusa.com | allardusa.com