SELECTION[®]

35206
RIGID

35204
35205
SOFT

SELECTION

SELECTION

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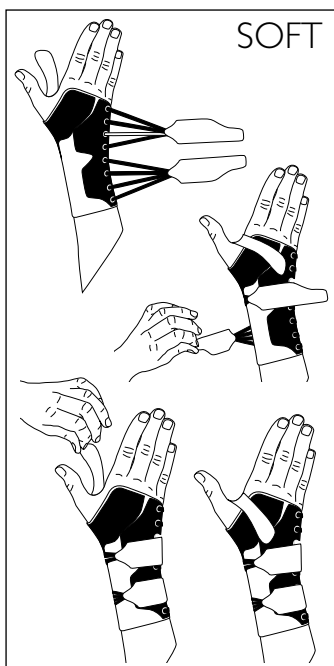
35204, 35205, 35206 SELECTION WRIST SUPPORT SOFT & RIGID

Carefully read this instruction sheet before use!

Intended use/Indications

Selection Wrist Soft

These products are intended for when the wrist needs support and off-loading in cases such as rheumatic disorders, soft tissue injuries or with degenerative changes.

Selection Wrist Rigid

These products are intended for wrists in need of support and off-loading of the first CMC-joint, in cases such as rheumatic disorders, soft tissue injuries or with degenerative changes.

Indications:

Inflammations, work related strain injuries or when rheumatic and/or degenerative changes are present.

Contraindications

No known.

Adjustments

These products should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your fit on a daily basis.

Material Specifications

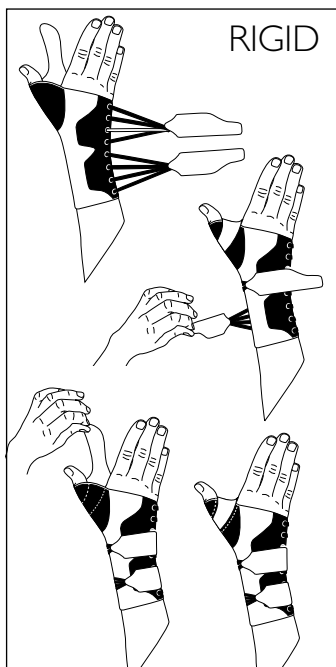
This product is made from polyester, nylon and elastan. Straps are made from plastic. This product is not flame retardant.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

1. Check that the support is complete and without damage.
2. Open the Velcro straps and release the lacing.
3. Pull the orthosis on like a glove.
4. Secure the two straps around the wrist.
5. Secure the thumb strap.
6. Adjust for comfort.

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Camp Scandinavia May2021 ©
Camp Scandinavia AB, Karvingatan 38